


verilux[®]
The Healthy Lighting Company™

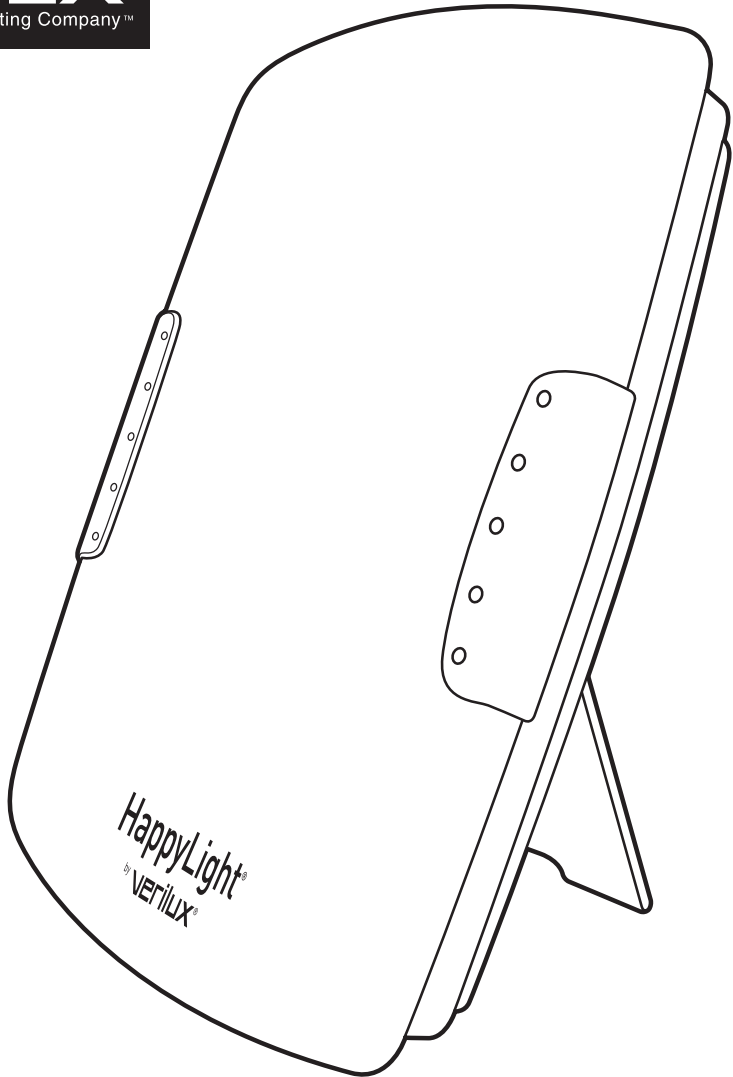

HappyLight[®] | Deluxe
Sunshine Supplement Light System

HPLD



Dear Customer,

Thank you for purchasing the Verilux® HappyLight® Deluxe Sunshine Supplement Light System. You have received an innovative and quality manufactured product, backed by a three-year limited warranty. As a Verilux® customer, your satisfaction means everything to us. We look forward to serving you now and in the future.

Many other healthy lighting products are available through our catalog and online. Visit us on the web at www.Verilux.com to request a free catalog and learn more about all our quality Verilux® products, or call our toll-free number 1-888-544-4865.

Have a Bright Day!

A handwritten signature in black ink that reads "Nicholas Harmon". The signature is written in a cursive, flowing style.



Nicholas Harmon
President and CEO

GENERAL PRECAUTIONS

- Do NOT dismantle.
- Do NOT place near flammable or combustible items.
- Do NOT alter power cord.
- Unplug from electrical outlet during lightning storms or when unused for long periods of time.
- Unplug from electrical outlet before cleaning or changing bulbs.
- For indoor use only.

Main Parts & Controls

- A. Lens Cover
- B. On/Off Switch
- C. Wall Mount Bracket
- D. Carrying Handle
- E. Built-in Stand
- F. Power Cord

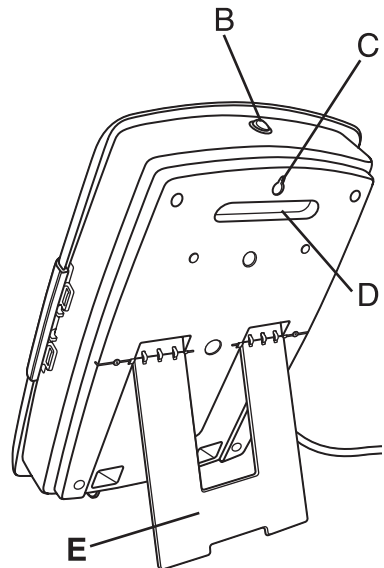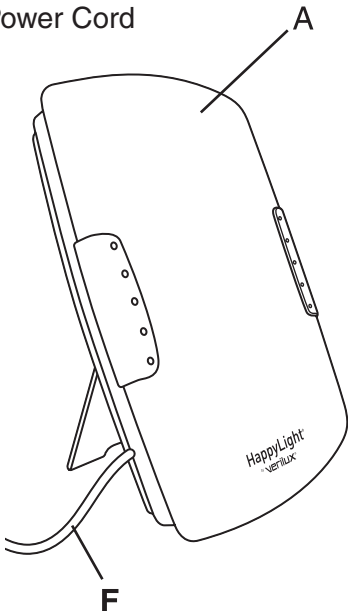

IMPORTANT SAFETY INSTRUCTIONS

Thank you for purchasing the Verilux® HappyLight® Deluxe Sunshine Supplement Light System. Please read the following safety information before using.

IMPORTANT:

- Verilux® does not make any medical claims. This product is not intended to treat or prevent any illness.
- It is recommended that you consult a medical professional before using the HappyLight®.
- Before plugging in the HappyLight®, check to make sure the voltage corresponds with the main voltage of your location (120 volts).
- Do not use the HappyLight® outside or in wet locations such as bathrooms. It is rated for indoor, dry use only.
- Children should be supervised around the HappyLight®.
- Not for use with light dimmers, timers, or motion detectors.
- This product may cause interference with radios, cordless telephones, or devices that use a wireless remote control, such as televisions. If interference occurs, move the product away from the device, plug the product or device into a different outlet, or move the product out of the line of sight of the remote control receiver. This product complies with part 18 of FCC.
- The light bulbs may become loose during transit. To reinstall the light bulbs, see the bulb replacement instructions.
- The Verilux® HappyLight® is an electrical device.

DO NOT USE NEAR WATER

It is advisable that you contact a doctor if:

- You are hypersensitive to light and/or experience eye discomfort.
- You are currently taking certain light sensitive drugs such as Tetracycline.
- There is no improvement after using the HappyLight® for two weeks, or if you feel uncomfortable.
- You have a history of eye disease.
- Do not treat the genitalia.
- Do not use if you are pregnant.
- If your skin responds negatively, discontinue use and consult a dermatologist.

IMPORTANT SAFETY INSTRUCTIONS

THIS PRODUCT MUST BE GROUNDED.

This product is equipped with a cord having an equipment–grounding conductor and a grounding plug. The plug must be plugged into an appropriate outlet that is properly installed and grounded in accordance with all local codes and ordinances.

DANGER

Improper connection of the equipment–grounding conductor can result in a risk of electric shock. Check with a qualified electrician or serviceman if you are in doubt as to whether the product is properly grounded. Do not modify the plug provided with the product – if it will not fit the outlet, have a proper outlet installed by a qualified electrician. This product is for use in a nominal 120 volt circuit, and has a grounding plug that looks like the plug illustrated in sketch (Fig. A). A temporary adapter that looks like the adapter illustrated in Fig. B may be used to connect this plug to a 2-pole receptacle as shown in Fig. B, if a properly grounded outlet is not available. The temporary adapter should be used only until a properly grounded outlet (Fig. A) can be installed by a qualified electrician. The green colored rigid ear, or lug extending from the adapter must be connected to a permanent ground such as a properly grounded outlet box cover. Whenever the adapter is used, it must be held in place by a metal screw.

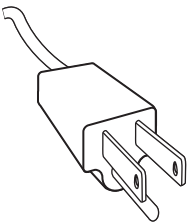


Figure A

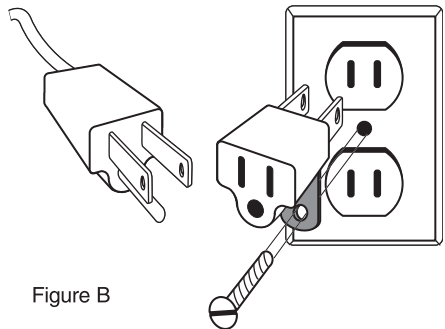
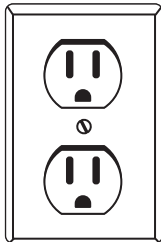


Figure B

INSTRUCTIONS FOR USE

To use the HappyLight® Deluxe Sunshine Supplement Light System:

1. Plug into properly grounded outlet.
2. Turn the unit on using the on/off switch. Allow a few minutes at the start of each session for bulbs to achieve full illumination.
3. Position yourself facing the HappyLight® (Figure 1). Light therapy consists of sitting close to the Sunshine Simulator with eyes open. Do not stare directly into the unit. While staying oriented toward the light feel free to engage in other activities such as reading, checking emails, hobby making & crafting, sewing, needlepoint, painting, or working at a computer/desk.
4. Time Exposure Recommendation for 10,000 Lux treatment:

DISTANCE FROM UNIT	TIME
27 inches	Approx. 2 Hours
18 inches	Approx. 1 Hour
9 inches	Approx. 1/2 Hour

Usage time depends upon your body and how it responds to light. Typical exposure is 30 minutes per day.

The ideal exposure time is in the morning, soon after waking. You should experience results within a week of daily exposure. It is recommended that you limit your exposure to the light during the evening, as it might effect your ability to fall asleep.

Each person's sensitivity to light is different. It is important to establish a distance and schedule that is comfortable for you. We recommend you use the light regularly and at about the same time each day. It is not necessary to sit in front of the unit continuously. You can divide the time into several mini sessions. Continue to use the HappyLight® daily until you feel an improvement in your mood and activity level.

IMPORTANT: *Tinted glasses will reduce the amount of light reaching your eyes.*

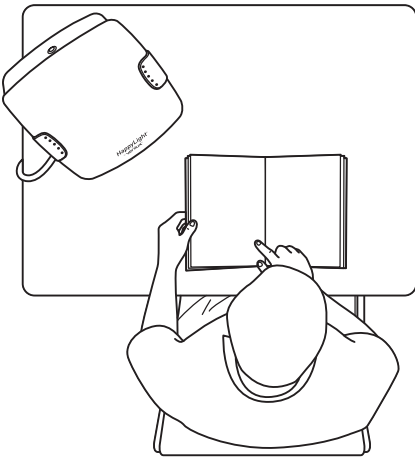


Figure 1

INSTRUCTIONS FOR USE

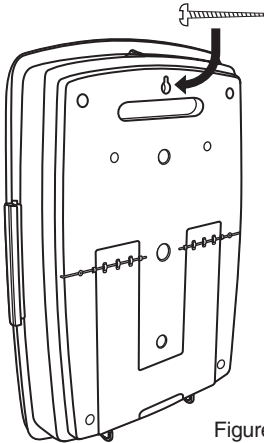


Figure 2

Hanging the HappyLight®

Place the screw (included) into the wall leaving approximately 3mm between the screw head and the wall. Be sure to use the included anchor if mounting into sheet rock or other soft material. Insert the screw head into the hole located in the upper part of unit. (Figure 2).

Standing the HappyLight® on a Table Top

Unfold the stand on back of lamp (Figure 3) and place the HappyLight® on a stable, flat surface.

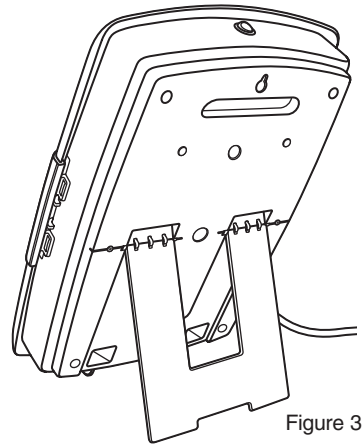


Figure 3

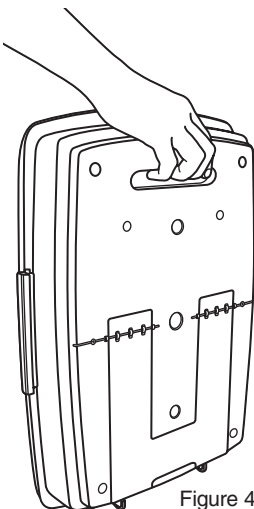


Figure 4

Transporting the Unit

The HappyLight® is very light and easy to carry. There is a convenient handle located on the back (Figure 4).

BULB REPLACEMENT INSTRUCTIONS

Step 1: Opening the Lens Cover

Place the HappyLight® on its back on a large flat surface. Press down gently on the center of the lens cover. (See Figure 5). This releases the locking tabs located under the grey handles. Open the lens cover by grasping the grey handles with your hands. Remove the lens and gently place it next to the unit with the logo face up.

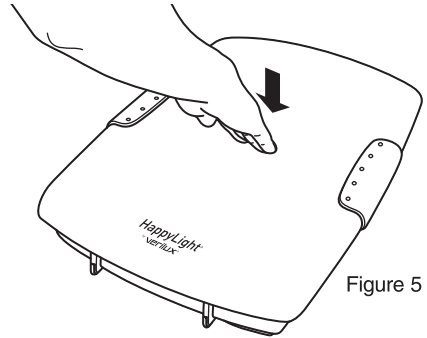


Figure 5

Step 2: Opening the Bulb Clips

Locate the clear plastic clips across each fluorescent tube (Figure 6). Put a finger under the clear plastic clip on the outer side and pull up (See Figure 7). The clip should pop open. Repeat on other side.

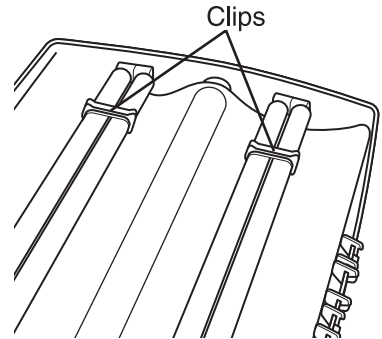


Figure 6

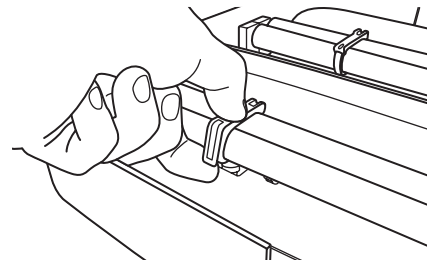


Figure 7

BULB REPLACEMENT INSTRUCTIONS

Step 3: Removing Bulbs

Gently lift the fluorescent tube up and out of the clip holder. Grasp the tube by the white plastic base with the Verilux® logo and remove the tube from the socket by pulling the tube towards the black cushion pads. (See Figure 8). Repeat removal of the second bulb.

Step 4: Installing New Bulbs

Ensure the HappyLight® is on its back on a large flat surface. Hold the fluorescent tubes by the white plastic base, not the glass. Ensure the Verilux® logo is facing up. Line up the pins on the base of the fluorescent tube with the holes in the socket. Gently push the bulb into the socket until it snaps securely in place. Repeat on other side.

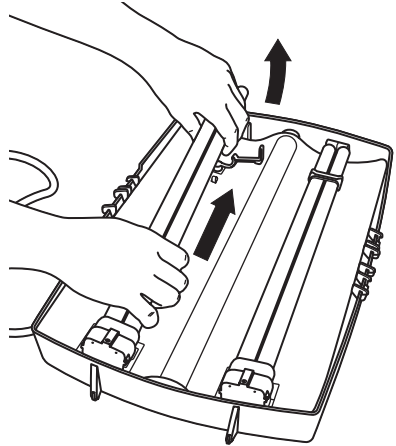


Figure 8

Bulb Disposal: All fluorescents contain some amount of mercury. Some states have special requirements regarding their disposal. Please go online to: www.lamprecycle.org or call 1-866-300-5606 for more information.

BULB REPLACEMENT INSTRUCTIONS

Step 5: Closing the Bulb Clips

Ensure the plastic bulb clips are centered on the white protective bands. Gently push on the upper portion of the fluorescent tube until it slides snugly into the clear plastic clip. Close the clip over the tube. The clip will click when fastened. Repeat on other side. (Figure 9).

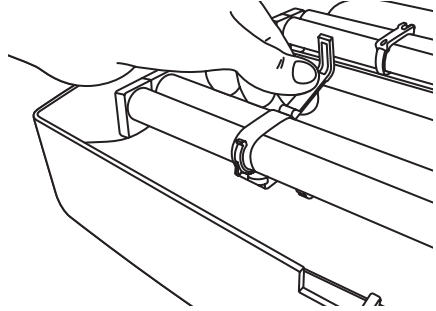


Figure 9

Step 6: Replacing the Lens Cover

Align the lens cover over the open face of the unit with the HappyLight® logo at the bottom of the lamp. Place your thumbs one inch beyond the grey handles towards the center of the unit while your other fingers grasp under the lip of the grey handles (see Figure 10). Pull outward with your fingers while pushing down on the lens with your thumbs. Release your fingers to snap the lens into place.

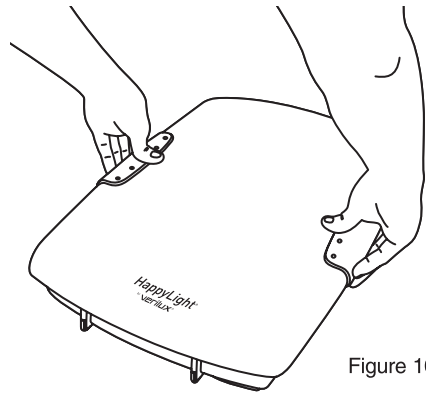


Figure 10

Plug your HappyLight® into an electrical socket and turn it on to ensure the bulbs were properly installed into their sockets. If the bulbs do not illuminate, ensure the bulbs are fully seated into the sockets.

CARE & CLEANING

Your HappyLight® Deluxe Sunshine Supplement Light System is made from high quality materials that will last for many years with minimum care. You may want to periodically clean the inside of the lens and outside parts, using a mild non-abrasive cleaner and soft cloth. When cleaning, make sure you have turned off and unplugged the unit, and allowed sufficient time for the unit to cool.

DO NOT USE SOLVENTS OR CLEANERS CONTAINING ABRASIVES, OR AMMONIA BASED CLEANERS

Cleaning the lens is important for maintaining optimum light output. Periodically remove the lens (refer to lens removal instructions) and clean with soft damp cloth. All liquid should be properly wiped away and dried prior to restoring power.

TROUBLESHOOTING

Light flickers.

If bulb flickers when turned on, the bulb is not seated fully into the socket. If this occurs, repeat the bulb installation process applying additional force to ensure that the bulb is inserted fully into the socket.

Does not illuminate.

Make sure that the bulbs are secured into the sockets. Sometimes in shipping they will come loose. Try a different grounded outlet. In many cases, this will correct the problem.

IMPORTANT – Grounded outlets must be used for proper function of the unit. If both bulbs do not illuminate when you apply power, this may indicate an internal problem. Please call Verilux® Customer Service at 1-888-544-4865. See Warranty section.

Only one bulb illuminates.

If only one bulb illuminates, it is most likely a bad bulb. Try swapping bulbs and turn on again. If the same bulb fails to illuminate in the swapped position, the bulb needs to be replaced.

Bulb blackening at the ends.

This is a common phenomenon with most fluorescent tubes as they age. The black areas don't affect operation except to slightly reduce the amount of light available since the phosphor in that area is dead. Bulb replacement is recommended.

NOTICE – When returning a unit, be aware that damage during shipping is the responsibility of the purchaser and will void any warranty claim. We recommend that you purchase insurance when shipping this unit or any parts. Be sure to return product in original packaging.

Three-Year Limited Warranty

ATTENTION! ONCE OPENED, PLEASE **DO NOT** RETURN THIS PRODUCT TO THE STORE WHERE IT WAS PURCHASED FOR REPAIR OR REPLACEMENT!

Many questions may be answered by visiting www.Verilux.com, or you may call our customer service department at 1-888-544-4865

This limited warranty is provided by:

Verilux, Inc., 340 Mad River Park, Waitsfield, VT 05673

Verilux® warrants this product to be free from defects in material and workmanship for a period of three years from the date of the original retail purchase. During the limited warranty period, Verilux, Inc. will, at its option, repair or replace defective parts of this product, at no charge to the customer, subject to the following limitations. This limited warranty does not include any postage, freight, insurance or delivery fees. This warranty does not cover damage, defect or failure caused by or resulting from accident, external destruction, alteration, modification, abuse, misuse, or misapplication of this product.

If, during the first Three years of ownership, this product fails to operate properly, it should be returned as specified below:

Please pack the defective product in its original protective packing and box, with a copy of your sales receipt, along with \$6.95 (check or money order to cover shipping and handling), and send to:

For U.S. Postal Service Only:

**Verilux Warranty
Service Center, c/o WTB**

P.O. Box 119, West Redding, CT 06896

For Fed Ex & UPS Only:

**Verilux Warranty
Service Center, c/o WTB**

32 Henry Street, Bldg 17A, Bethel, CT 06801

Please allow 4-6 weeks before you receive the repaired or replaced product.

ATTENTION: When returning a product, be aware that damage during shipping is the responsibility of the purchaser and will void any warranty claim. We recommend that you insure the shipment by shipper when returning this product or any parts. Be sure to return product in original packaging.

***Note:** Verilux® recommends using a quality surge suppressor on all electronics equipment. Voltage variations and spikes can damage electronic components in any system. A quality suppressor can eliminate the vast majority of failures attributed to surges and may be purchased at electronics stores.*

Due to ongoing improvements, actual product may have slight variations from the product described in this manual.

Please visit our web site at: www.Verilux.com

verilux®
LIFE IN A BETTER LIGHT®

340 Mad River Park, Waitsfield, VT 05673